

## USING PETSPEMF PAD FOR THE FIRST TIME

1



### 1. CHARGE

Charge your Petspemf Pad for a minimum of 2.5 hours when using it for the first time. Petspemf Pad is unlocked when plugged into the charger.

2



### 2. DOWNLOAD THE APP

Download the Petspemf App to your mobile device or tablet from the iOS App Store or Google Play store.



Search for Petspemf

3



### 3. START THE APP

Press START and accept the Terms and Conditions. Make sure to have Bluetooth on your device turned ON and follow the step-by-step User Guide. If you are using an Android device, you will be asked to enable location services using the app.

4



### 4. WAKE UP THE DEVICE

Double tap the control box to wake up the device (confirmed by a double vibration) - at this point the device is ready to use.

5



### 5. CONNECT

Click the CONNECT button in the app and wait until Petspemf Pad pairs with your device.

6



### 6. START SESSION

Once you select a program, set your duration and the session will begin (followed by single vibration). To stop a session, either press the STOP button in the app or double tap the control box (followed by double vibration).